


A Great Smile

MAKES ALL THE DIFFERENCE

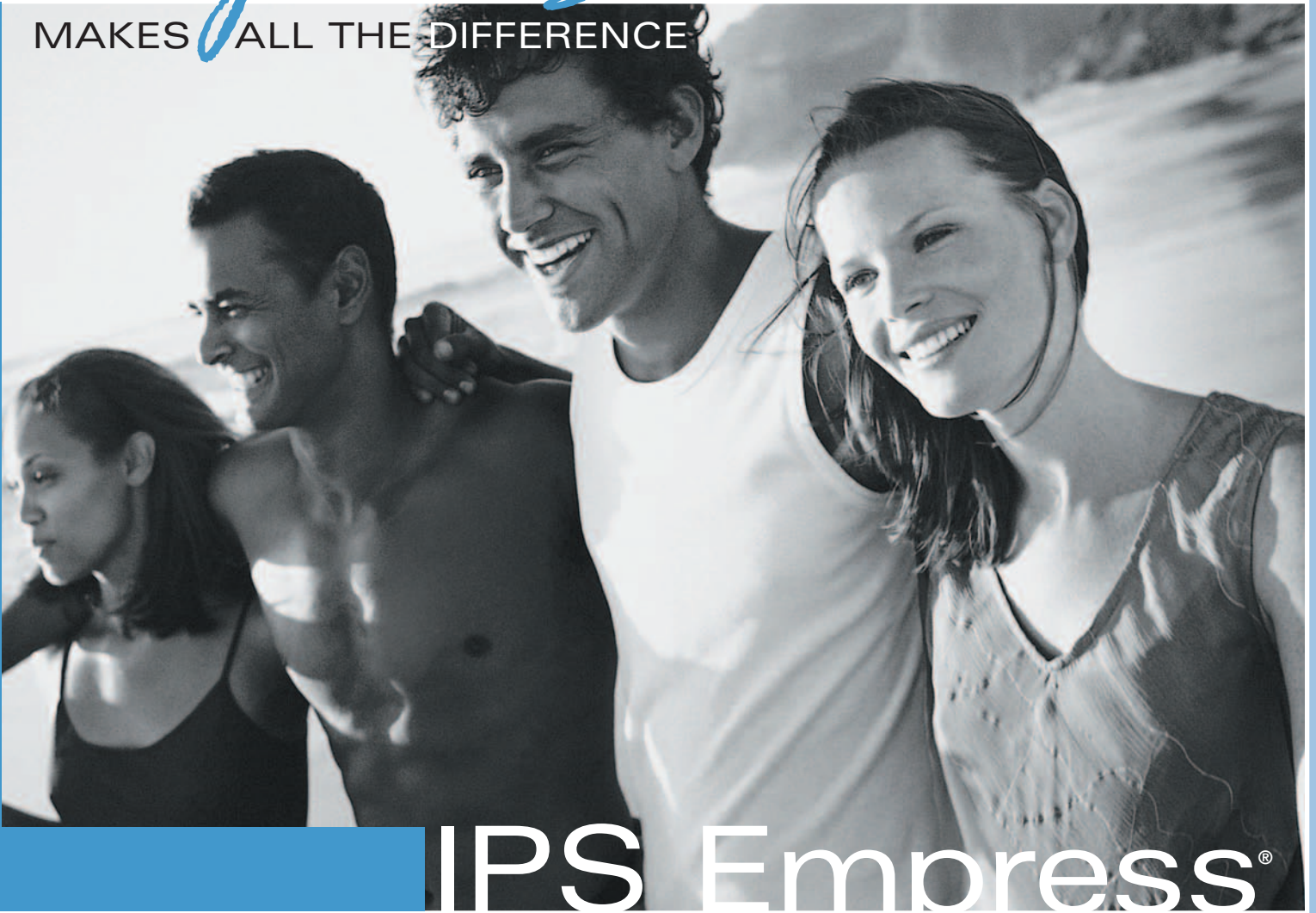

IPS Empress®

The Ultimate in Metal-Free Esthetics

With IPS Empress you can offer your patients an all ceramic restorative material with 15 years of clinical success. IPS Empress combines outstanding esthetic results with excellent fit and strong wear characteristics. Make IPS Empress your all ceramic of choice.

- **Over 25 million IPS Empress restorations placed worldwide.**
- **IPS Empress restorations wear like enamel.**
- **Indicated for inlays, onlays, veneers and crowns.**
- **IPS Empress restorations are adhesively cemented.**

Call Today and Find Out How
IPS Empress can make a Difference
in Your Practice.



IPS Empress Available From:


Laboratoire de porcelaine dentaire Longueuil
141-P Boulevard de Mortagne
Boucherville, Qc, J4B 6G4
(450) 674-4827 / 1-800-567-4827
info@lablongueuil.com
www.lablongueuil.com

IPS Empress® Preparation Guide


Anterior

Posterior

ANTERIOR CROWN PREPARATION

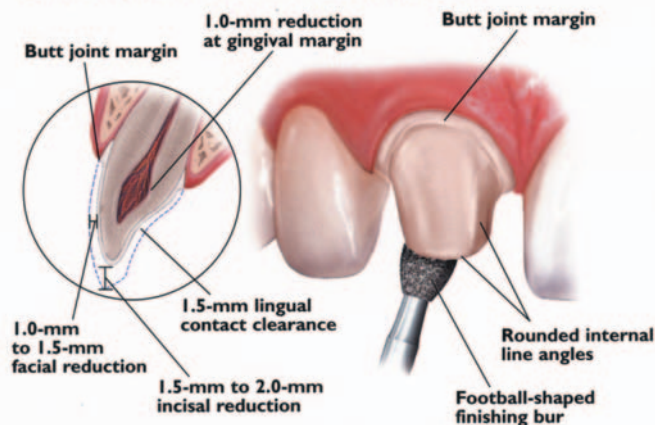


POSTERIOR CROWN PREPARATION

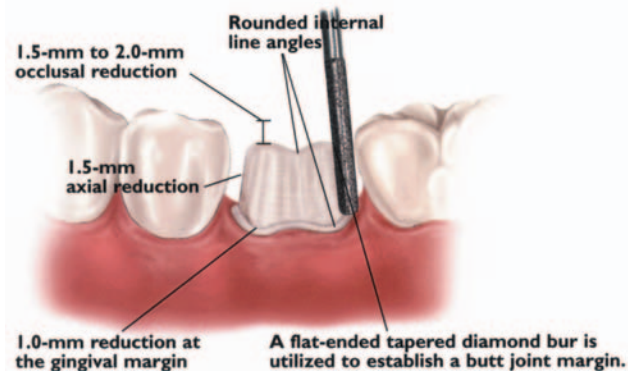


UNIFORM FACIAL PREPARATION

A medium grit, round-ended, diamond bur is used to remove a uniform thickness of facial enamel by joining the depth-cut grooves.

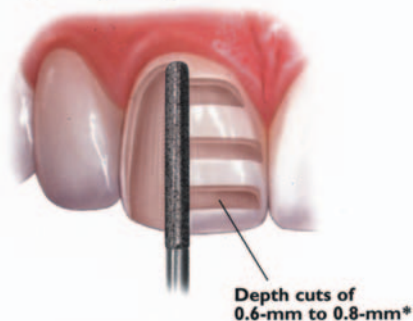


INLAY PREPARATION

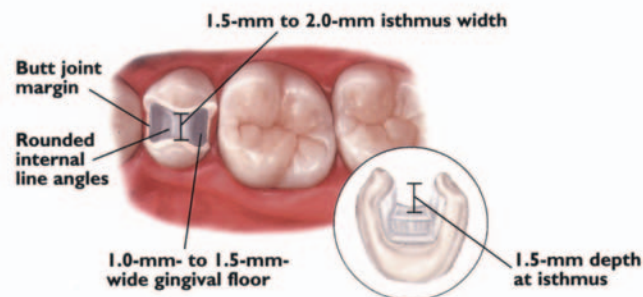


INCISAL PREPARATION

The diamond bur is angled to bevel back the incisal edge.

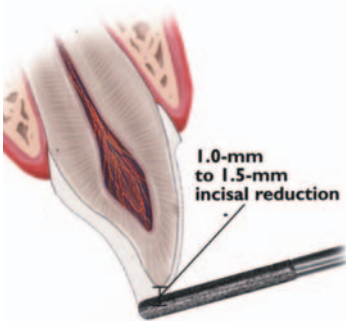


CHAMFER MARGINS

Correct preparation of the chamfer margins interproximally allows the appropriate bulk of porcelain.

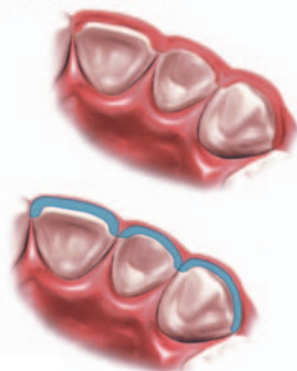

ONLAY PREPARATION

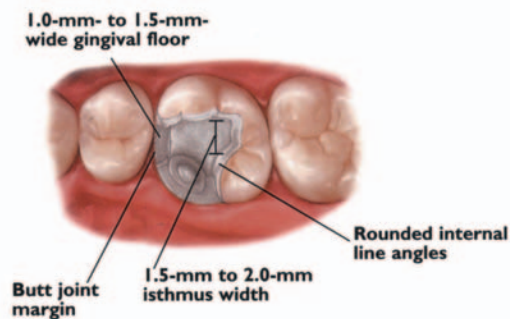

IPS Empress Available From:
 Laboratoire de porcelaine dentaire Longueuil
 141-P Boulevard de Mortagne
 Boucherville, Qc, J4B 6G4
 (450) 674-4827 / 1-800-567-4827
 info@lablongueuil.com
 www.lablongueuil.com